# **GUARDIAN®**

Family-owned. Trusted by hospitals for over 50 years.  
Entreprise familiale. Marque approuvée par les hôpitaux depuis plus de 50 ans.  
Empresa familiar. Cuenta con la confianza de hospitales por más de 50 años.

## **FOLDING COMMODE** USER INSTRUCTIONS AND WARRANTY

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**LIFETIME**  
LIMITED  
WARRANTY

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# MAINTENANCE

- Periodically, inspect the commode for damaged or loose components, and ensure that all parts are secure.
- Check that there are no obstructions to normal, safe operation.
- To maintain appearance and good hygiene, clean periodically with mild soap and rinse with warm water.
- Wipe bath seat dry after each use.

NOTE: If any part or mechanism appears to be unsafe or damaged, DO NOT USE THE PRODUCT. If you experience problems, please contact Medline at 1-800-MEDLINE.

# ASSEMBLY:

1. Before assembling seat and lid, allow toilet seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled.
2. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets). Push down on seat until clamps snap firmly into place over frame rail.
3. Repeat the same process for the lid. Note: (G1-202EX4 and G1-202ELX4 do not come with lid)
4. When removing the seat or lid, always remove lid first and then the seat. To remove, grasp the back of the lid and/or seat and lift.
5. Install commode bucket by dropping into bucket frame.

Lift bucket from frame to remove for emptying and cleaning.

## Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When your desired height is reached, lock legs into place by aligning buttons with holes until they “snap” into place (see Figure 1).
2. Make sure legs are locked into place by pushing down or by pulling up on legs.

**Figure 1**

## CAUTIONS:

1. Toilet seat must be in down position before use. Failure to do so may result in possible injury or breaking the commode.
2. Do NOT install commode without first reading and understanding this assembly and instruction guide. If you cannot understand these instructions, contact a healthcare dealer or technician before attempting assembly. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact Medline at 1-800-MEDLINE.
4. Make certain that the height adjusting buttons snap into place and fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved (see Figure 1).
5. Ensure ALL screws, nuts and/or bolts are tightened.
6. Check the tips on the leg extensions for rips, wear or if they are missing. Immediately replace missing or damaged tips.
7. All four leg extensions with tips MUST touch the floor simultaneously at all times.

8. Users with limited physical strength and/or mobility should be supervised and/or assisted while using the commode.
9. DO NOT exceed the user weight capacity of 350 lbs (159 kg).
10. Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.
11. Evenly distribute weight between the arm supports.

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## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Industries, Inc.

---

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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