**2 BUTTON  
FOLDING WALKER  
USER INSTRUCTIONS  
& WARRANTY**

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# Safety Instructions

**For safety, these precautions must be taken when using this product.**

**Before each use make sure:**

- Walker is adjusted to correct height.
- Locking mechanisms and spring buttons are fully engaged.
- Tips are in good condition. Replace tips immediately when worn or missing.
- Make sure the walker sits level and all legs are adjusted to an equal height.
- Use only as instructed by your healthcare provider.

Always use extreme care when using a walker. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

Use wheels on front legs of walker only with the exception of Rear Wheel Brakes.

Do not "rack" the walker by pushing one side forward and then the other side without lifting the walker.

Use Guardian accessories only. Use of other accessories may result in improper fit and risk of fall and injury.

Do not use on stairs unless otherwise instructed by a professional.

The use of walkers in water (showers, bathtubs, pools, etc.) may corrode the working mechanisms of your walker. The use of a corroded walker may result in damage to the walker and/or injuries to the user.

When transferring from a wheelchair to a walker, be certain wheelchair wheels are locked.

Do not exceed the maximum weight capacity of your walker.

Do not use your walker if you have any handicaps or disabilities or using medications, etc. that would impair or interfere with safe usage of this product.

**Failure to follow the above instructions may result in serious injury.**

**Tips on this product contain dry natural rubber.**

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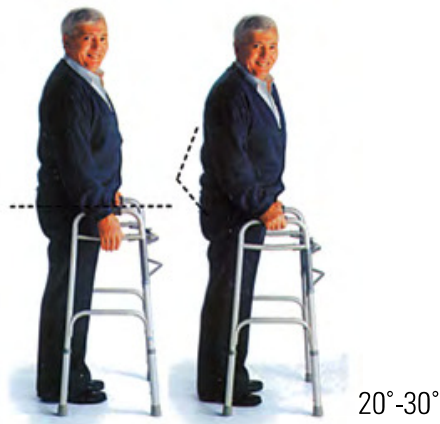
Clean the grips with a mild soap detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

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**5.**

## **WHEELCHAIR TO WALKER TRANSFER**

(no specific lower extremity injury)

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With the wheelchair wheels locked, push down on the armrests and transfer one hand at a time to the walker.

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With the wheelchair wheels locked, push down on the armrests and transfer the left hand to the walker, followed by the right hand.

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**WARNING!** Be sure all locking mechanisms are secure before using the walker.

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**Mecanismos de Bloqueo de Patas**

## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada  
1-800-396-6996  
www.medline.ca | Canada@medline.com

Medline Mexico  
01-800-831-0898  
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2014 Medline Industries, Inc.

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