GUARDIAN[®]

Family-owned. Trusted by
hospitals for over 50 years.

**BARIATRIC ROLLATOR
HEAVY DUTY
ROLLING WALKER
WITH WHEELS
USER INSTRUCTIONS
& WARRANTY**

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**LIFETIME
LIMITED
WARRANTY**

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Product is not made with
Natural Rubber Latex.

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Safety Instructions

GENERAL CAUTIONS:

- **DO NOT** use this product without first reading and understanding the instructions contained in this booklet. If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technical personnel before use – otherwise serious bodily injury and damage may occur.
- Consult with your doctor, physical, or occupational therapist or other qualified professional to assure the proper height adjustment.
- **DO NOT** use the rollator as a wheelchair.
- **DO NOT** attempt to move the rollator while you or anyone is sitting on the seat.
- **DO NOT** self-propel the rollator while seated.
- Serious injury to you and/or damage to the rollator walker frame or wheels may result from improper use.
- **DO NOT** use the seat to transport people or objects. **DO NOT** use the seat to carry or move anything.
- **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- **DO NOT** use the rollator as a scooter or a knee walker. The rollator was designed to only be used as a walking aid, not hopping or any other type of mobility.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result: Maximum weight capacity: 400 lbs (181 kg)
- Before each use make sure the rollator frame is fully opened.
- Check the brakes for proper operation. Make sure that all parts are secure and that moving parts are in good working order. Wheels should roll and not wobble.
- All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.
- The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- **DO NOT** sit on the walker if ground is sloped or uneven.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- **DO NOT** perform any adjustments to the rollator while it is in use.
- Only use accessories and spare parts authorized by Medline Industries, Inc.
- **DO NOT** push rollator while brakes are enabled. Brakes are used to keep the wheels from rolling. But it may not prevent the rollator from moving if it is pushed.
- **DO NOT** use the rollator to provide assistance while rising from a lower position. If you use it to pull yourself up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- Inspect the handle bar grips prior to use. Assure the plastic parts are still functional if they are exposed to extreme temperatures (above 100°F (38°C) or below 32°F (0°C) high humidity and/or become wet.
- **DO NOT** twist hand grips as they are not meant to be twisted. Failure to do this could result in a fall.
- **DO NOT** hang anything from the frame of the rolling walker other than the basket included or the tote bag accessory. The basket has a weight limitation of 11 lbs (5 kg). The tote bag has a weight limitation of 10 lbs (4.5 kg). Items should be placed completely in the basket or tote bag and should not protrude. This may cause the rollator to tip, resulting in bodily injury or rollator damage.
- The backrest is intended to provide back support as you sit on the seat. The backrest is not intended to support full weight. Avoid leaning while sitting as this may result in fall.

Assembling The Rollator

Figure 1: Assembled Rollator

Rollator Components

- (1) Rollator frame
- (2) Front wheels
- (2) Handlebars
- (2) Rear wheels
- (1) Backrest with push-buttons
- (4) Round Knob Screws
- (2) Triangle Knob Screws
- (1) Basket

Rollator Components

- A.** Handgrips
- B.** Adjustable Handlebar
- C.** Padded Backrest
- D.** Brake Cables
- E.** Padded Seat
- F.** Support Rod
- G.** 8" (203 mm) Front Wheels
- H.** 8" (203 mm) Rear Wheels
- I.** Folding Bracket
- J.** Round Hand Screws
- K.** Triangle Knob Screw
- L.** Round Knob Screw
- M.** Brake Shoe/Brake Blade
- N.** Hand brake

Assembly Instructions

IMPORTANT: Read all operating and safety instructions before using your rollator.

Note: You may wish to request a set up and operating demonstration with your local Medical Equipment provider or pharmacy.

1. Carefully remove the rollator components from the shipping carton. Refer to diagram (Figure 1) as needed.
2. Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the basket) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at **1-800-MEDLINE**.
3. Lay the rollator down so that the seat is facing the floor. Cut the plastic ties from the rear wheels and unfold the legs. Straighten the legs until the rollator unfolds.
4. See Figure 2 for the basic layout of the rollator.

Fig 2

5. Check all 4 legs to see if there is a plug in the end of any of the tubes. If there are, remove them (Fig 3 and Fig 4).

Fig 3

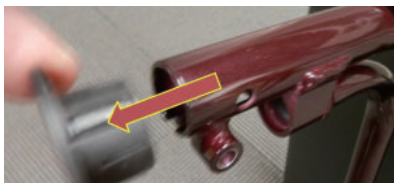
Fig 4

6. Flip the rollator up on its side as in Figure 2 above.

7. Insert the front wheel (G) into the front wheel receptacle (Fig 5 and Fig 6).

Fig 5

Fig 6

8. Make sure the bolt on the side of the insert goes all the way into the slot (Fig 7).

Fig 7

9. Insert the round hand screw (J). Make sure the threaded portion of the round hand screw is underneath the bolt in the side of the insert (Fig 8).

Fig 8

10. When installing the rear wheels (H), the wheel and brake shoe must be oriented towards the outside of the rollator (Fig 9).

Fig 9

11. Insert the rear wheel (H) into rear wheel receptacle. The rear wheels have the brake shoe attached, which is attached to the cable and the adjustable handlebar (Fig 10 and Fig 11).

Fig 10

Fig 11

12. Make sure the bolt on the side of the insert goes all the way into the slot (Fig 12).

Fig 12

13. Insert the round hand screws (J). Make sure the threaded portion of the round hand screw is underneath the bolt in the side of the insert (Fig 13).

Fig 13

14. Take the adjustable handlebar (B) that is attached to the rear wheel just installed in step 9 and insert it into the handlebar receptacle on the same side as the wheel is (Fig 14). The brake cable should be on the outside of the rollator, running down along the rear leg. Align 1 of the holes in the handlebar to the hole in the frame. Insert the triangle knob screw (K), **HAND TIGHT ONLY** (Fig 15).

Fig 14

Fig 15

15. Attach the cable to the cable clip on the rear leg (Fig 16). Extra length in brake cable should loop towards the front of the rollator (Fig 17).

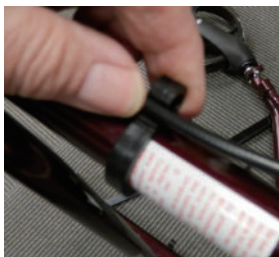
Fig 16

Fig 17

16. Turn rollator over on other side and repeat steps 7-13 above.

17. Stand the rollator upright. Check that all 4 wheels are resting evenly on the floor (Fig 18).

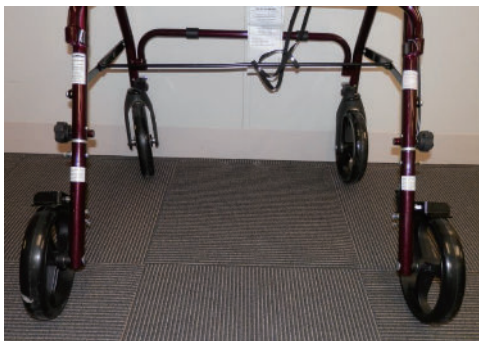
Fig 18

18. Adjust the height of the adjustable handlebars (B) for proper fit.

To determine proper height, stand upright behind the rollator with shoulders relaxed, arms hanging at your side and eyes looking straight ahead. Standing in this position, the handgrips (A) of the rollator should be adjusted to a height even with the wrists. This will position the arms at an approximate 20° to 30° bend when using the rollator. Secure the handlebars at the closest hold to the desired height. Insert and tighten the triangular hand screws until secure. (Important: Hand Tighten Only!)

19. The basket can be mounted either under the seat (Fig 19) or in front of the seat (Fig 20).

Fig 19

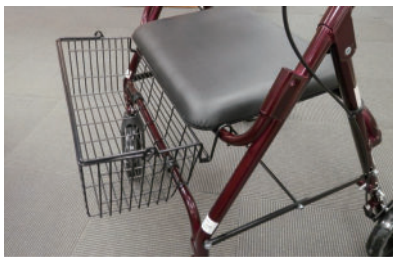
Fig 20

Operating Instructions

Opening the Rollator

Unfold the Medline rollator by pushing down on the support rod (F) to spread the frame apart until the folding brackets (I) are in full extension. Push the seat down into place.

Folding the Rollator

Remove the basket from under the seat. Pull up on the folding bracket tie-bar on the support rod (F) until the frame is fully collapsed.

Checking the Brakes

- Before each use, check to make sure the brakes are working properly.
- Applying pressure on brake handles using a pulling up motion the brakes will stop the wheels until the brake handle is released (Fig 21).
- To lock the brake on, push down on the brake handles until they snap into a lower position (Fig 22). The brake handles will stay in the lower position and the brakes will remain locked on. To release the lock, simply pull up on the brake handles. If the brakes have been adjusted but the rear wheels on the rollator move while the brakes are in the locked position, **DO NOT** use the rollator. Please call our toll-free customer service number at **1-800-MEDLINE** for returns or repairs.

Fig 21

Fig 22

Using the rollator as a walker

For proper positioning and use of your rollator, imagine a line drawn across the back of the rear wheels. Part of one foot should be slightly ahead of that line as you walk.

Using the rollator as a seat

- Push the rollator into the position where you want to sit.
- **LOCK** the brakes **BEFORE** sitting.

To Operate the Brakes

- To slow down, squeeze the brake handles up.
- To lock the brakes, push down on the brake handles until you feel or hear the brakes click into locked position. If the brakes have been adjusted but the rear wheels on the rollator move while the brakes are in the locked position, **DO NOT** use the rollator. Please call our toll-free customer service number at **1-800-MEDLINE** for returns or repairs.
- To release the brakes from the locked position, simply pull up on the brake handles.

Safety Warnings: Also See Instructions

Before using the seat portion of your rollator, **ALWAYS** lock the brakes **FIRST**. **DO NOT** navigate the rollator while sitting on the seat. The brakes should always be locked when the seat is being used. Risk of fall and serious injury may result if brakes are **NOT** locked while the user is sitting on the seat.

Maintenance

Your rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure. Periodically test the brakes by both walking and squeezing the brakes and by stopping, pushing down on the brakes to a locked position, and then trying to push the walker. If the brakes do not lock the wheels, the brakes need to be adjusted. If adjustment does not tighten the brakes, **DO NOT** use the rollator.


Adjustment of Brakes:

(There are 2 adjustments for each brake – one at the handle and one at the brake):

When adjusting the brakes on the rollator, keep in mind if you adjust it too far one way or the other, the lock will not function properly. Please test thoroughly and tighten all parts completely prior to using rollator.

1. At the top of the rollator, find the adjustment nut which is located where the cable goes into the brake handle. This part is usually silver.
 - If the brake is too tight, at the handles of the rollator, check to see if there is a gap between the adjustment nut and thumb screw. If there is, turn the adjustment nut counter-clockwise until it is flush up against the thumbscrew (You may need pliers to get started). Then rotate both together clockwise until it is tight against the handle. . . this should loosen the brake. IF there is no gap and it is already tight against the handle, no further adjustment can be made here. . . Refer to Step 3 below.
 - If the brake is too loose (not locking), standing at the handles of the rollator, turn the adjustment nut and thumbscrew counter-clockwise.

(You may need pliers to get started). The thumbscrew and adjustment nut will push away from the handle and this will tighten the brake. Once the brake is adjusted properly, rotate just the adjustment nut clockwise until snug against the handle. There should now be a gap between the adjustment nut and the thumbscrew.

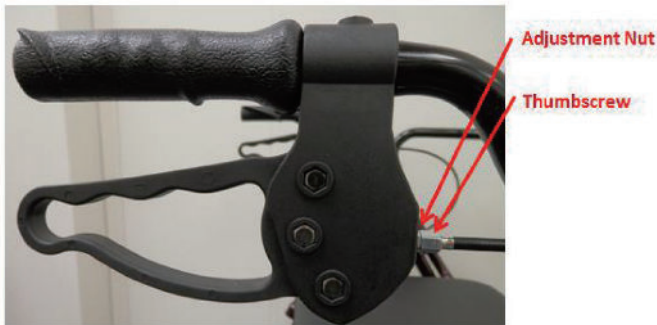2. At the bottom of the rollator, find the adjustment nut similar to the one located at the top. Hold the brake pad against the wheel. This will take the tension off of the cable.

- If the brake is too tight, check to see if there is a gap between the adjustment nut and thumb screw. If there is, pull up on the cable, and looking at the adjustment nut from the top, turn the Adjustment Nut counter-clockwise, screwing the adjustment nut to the Thumb Screw. Adjust to correct tension. IF there is no gap, no further adjustment can be made here. . . Refer to Step 3 below.
- If the brake is too loose (not locking), pull up on the cable, then, looking at the adjustment nut from the top, turn the Adjustment Nut clockwise, screwing the adjustment nut to the bracket. There will now be a gap between the Thumb Screw and the Adjustment Nut. Adjust to correct tension.

Keep your receipt!

Contact us at 1-800-MEDLINE, Monday – Friday 8am to 5pm CST if you encounter any problems or for information regarding repairs or maintenance.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN®

Medline Industries, Inc.
Three Lakes Drive
Northfield, IL 60093

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com | info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2018 Medline industries, Inc. Medline and Guardian are registered trademarks of Medline Industries, Inc.

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