GUARDIAN[®]

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ALUMINUM BATH SEAT WITH BACK

USER INSTRUCTIONS AND WARRANTY
Weight capacity 400 lbs

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REF G2-102KX4
G2-102KX1

MAINTENANCE

Periodic visual inspection of this bath seat is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE.

ASSEMBLY:

1. Open box and make sure it contains all parts described in this manual.
2. Place seat upside down on a flat surface so the four leg receptacles are facing upward.

Figure 1

3. Insert first leg into the leg receptacle by depressing pushbutton. When the leg is in place, the pushbutton will extend out and lock the leg in place. (Figure 1)
4. Repeat Step 3 for the remaining three legs.
5. Attach backrest to seat by inserting both back support tubes into the back support receptacle on the seat. The pushbuttons will snap out and lock the back support in place. (Figure 2)

Figure 2

Adjusting Leg Height

1. Before using the seat, make sure all legs are set to the same preferred height.
2. To do so, depress pushbutton and slide each leg to desired height by aligning with proper holes and letting the pushbutton snap into the correct hole to lock leg in place. (Figure 3)

Figure 3

3. Make sure legs are locked into place by pushing down and pulling up on each leg.

CAUTIONS:

1. Inspect all parts for possible damage before assembling. If there is any damage - DO NOT USE. Contact dealer/ carrier for further instructions.
2. Make sure that the height adjusting pushbuttons fully extend through the same corresponding hole of each leg extension. This ensures that the leg extensions are securely locked in place at an even height.
3. Make sure all screws, nuts and/or bolts are tightened.
4. Make sure the rubber tips are in place on the leg extensions and not ripped, worn or missing. Replace any missing or damaged tips.
5. The rubber tips of all four leg extensions MUST touch the floor simultaneously at all times.
6. NEVER stand on chair.

NOTE: To maintain appearance and promote good hygiene, clean periodically with mild soap and rinse with warm water. Wipe bath seat dry after each use.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN®

Medline Industries, LP,
Three Lakes Drive
Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com
www.info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca
Canada@medline.com

Medline Mexico
01-800-831-0898
medlinemexico.com
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2020 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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