GUARDIAN

Family-owned. Trusted by hospitals for over 50 years.
Entreprise familiale. Marque approuvée par les hôpitaux depuis plus de 50 ans.
Empresa familiar. Cuenta con la confianza de hospitales por más de 50 años.

DELUXE STEEL DROP ARM COMMODES

USER INSTRUCTIONS AND WARRANTY
Weight capacity 350 lbs (158 kg)

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REF G1-301DPX1
G1-301DX1
G1-301DX2

GENERAL WARNINGS

To ensure the safe assembly and operation of the commode, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.**

If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged, contact Medline at **1-800-MEDLINE | 1-800-268-2848**.
- **DO NOT** exceed weight capacity of 350lbs (158 kg).
- **DO NOT** make any adjustments while using the product.
- **DO NOT** attempt to use the arms to support full body weight.
- **DO NOT** stand on the commode.

MAINTENANCE

Periodic visual inspection of this commode is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at **1-800-MEDLINE | 1-800-268-2848**.

ASSEMBLY:

1. Remove contents from box. Depress pushbuttons and insert bucket frame section into first arm. The pushbuttons will snap into place when frame is securely in place. Note: Make sure the back of the bucket frame (the side with the backrest receptacles) is on the same side as the back of the armrest section.
2. Repeat for second arm section.
3. Insert backrest into backrest receptacles in commode bucket frame. The pushbuttons will snap into place when secure.
4. Align the padded or unpadded toilet seat clamps onto the rear frame rail. (Both clamps should sit outside the rail brackets). Push down on seat until clamps snap firmly into place. To remove, grasp the back of the seat and lift.
5. Install commode bucket by dropping into bucket frame. Lift bucket from frame to remove for emptying and cleaning.

LEG HEIGHT ADJUSTMENT

Before using the commode, make sure all legs are set to the same preferred height.

1. To adjust leg height, depress pushbutton and slide each leg to the desired height by aligning with proper holes and letting the pushbutton snap into the correct hole. (Figure 1)
2. Repeat until all legs are adjusted to the

Figure 1

preferred height.

3. Make sure legs are locked into place by pushing down and pulling up on each leg.

CAUTIONS:

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product.

Prior to use, ensure the following:

1. Toilet seat must be in the down position before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components and ensure all parts are secure.
4. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause a loss of balance and/or damage to the product.
5. Tips on the extension legs do not contain rips, cracks or are excessively worn. Immediately replace missing or damaged tips.
6. All four legs touch the floor simultaneously at all times.
7. User evenly distributes weight between the arm supports when transferring in and out of the commode.
8. Arms should be unlatched and dropped to assist in transfer on or off the device.

CLEANING

- Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
- **DO NOT** use abrasive powders, cleaners or scouring pads.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN[®]

Medline Industries, LP,
Three Lakes Drive,
Northfield, IL 60093 USA.

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com
www.info@medline.com

Medline Canada
1-800-268-2848
www.medline.ca
Canada@medline.com

Medline Mexico
01-800-831-0898
medlinemexico.com
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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