# GUARDIAN®

Family-owned. Trusted by hospitals for over 50 years.  
Entreprise familiale. Jugée fiable par les hôpitaux depuis plus de 50 ans.  
Empresa familiar. Cuenta con la confianza de hospitales por más de 50 años.

## TRANSFER BENCH BATH AID USER INSTRUCTIONS AND WARRANTY Weight capacity 400 lbs (181 kg)

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**LIFETIME**  
LIMITED  
WARRANTY

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# MAINTENANCE

Periodic visual inspection of this transfer bench is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE

## ASSEMBLY:

1. Remove contents from box making sure you have all parts described here.
2. Place both transfer bench seat sections upside down on a flat surface.
3. Slide smaller seat section into larger section by depressing the brass pushbuttons and inserting into the receptacles. Pushbuttons will pop out when secure. (Figure 1)

**Figure 1**

4. Remove black top from 4 leg receptacles.
5. Attach leg with the large suction cup (marked "A") by depressing the pushbutton and inserting into the receptacle with the matching "A" until the pushbutton snaps and locks into place. Repeat for second leg. (Figure 2)

**Figure 2**

6. Attach leg with small rubber tip (marked "B") by depressing pushbutton and inserting into the receptacle with matching "B" until the pushbutton snaps and locks into place. Repeat for second leg. (Figure 3)

**Figure 3**

7. Turn transfer bench right-side up. Attach armrest by depressing pushbuttons and sliding into the arm receptacle on the side of the bench. Make sure the arm arcs outward, not inward. (Figure 4)

**Figure 4**

8. To assemble backrest, insert support tube into backrest support by depressing pushbutton and sliding into backrest tube. Repeat for second support tube. (Figure 5)

**Figure 5**

9. Insert backrest into base by depressing pushbuttons and sliding support tubes through both receptacles. Note: Backrest must go through both sets of receptacles to become secure. (Figure 6)

NOTE: Back support is reversible to accommodate left or right tub entry. You can also change the seat depth of your transfer bench by pushing the pins under the seat and sliding the tubes forward until the pins lock into place.

**Figure 6**

10. Place legs with the large suction cups into bath tub, smaller feet outside the tub.

## **Adjusting Leg Height**

1. Before using the bench, make sure all legs are set to the same preferred height. When your desired height is reached, lock legs into place by aligning pushbuttons with holes until they "snap" into place.
2. Make sure legs are locked into place by pushing down and pulling up on each leg.

## **CAUTIONS:**

1. Inspect all parts for possible damage before assembling. If there is any damage - DO NOT USE. Contact dealer/ carrier for further instructions.
2. Make sure that the height adjusting pushbuttons fully extend through the same corresponding hole of each

leg extension. This ensures that the leg extensions are securely locked in place at an even height.

3. Make sure all screws, nuts and/or bolts are tightened.
4. Make sure the rubber tips are in place on the leg extensions and not ripped, worn or missing. Replace any missing or damaged tips.
5. The rubber tips of all four leg extensions **MUST** touch the floor simultaneously at all times.
6. **NEVER** stand on chair.

**NOTE:** To maintain appearance and promote good hygiene, clean periodically with mild soap and rinse with warm water. Wipe bath seat dry after each use.

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## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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---

**GUARDIAN**<sup>®</sup>

Medline Industries, Inc.,  
Three Lakes Drive, Northfield,  
IL 60093 USA.

---

Medline United States  
1-800-MEDLINE (633-5463)  
[www.medline.com](http://www.medline.com)  
[info@medline.com](mailto:info@medline.com)

Medline Canada  
1-800-268-2848  
[www.medline.ca](http://www.medline.ca)  
[Canada@medline.com](mailto:Canada@medline.com)

Medline Mexico  
01-800-831-0898  
[www.medlinemexico.com](http://www.medlinemexico.com)  
[mexico@medline.com](mailto:mexico@medline.com)

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2020 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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